



ResMed

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
Astral™ series




Astral big button quick setup guide

Getting started on ResMed Astral™

- 1 Touch screen to wake the device.



- 2 Touch "Unlock touch screen."

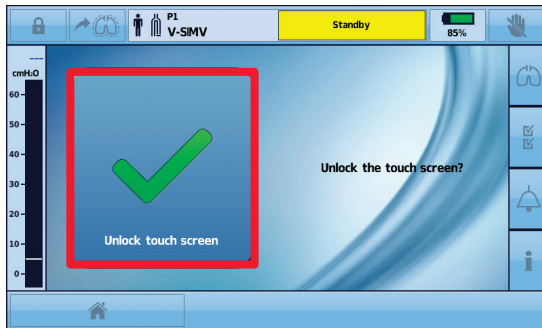

Starting therapy on Astral

- 1 Touch "Start vent" to begin therapy.




Stopping therapy on Astral

- 1 Touch and hold "Stop vent" until blue line fills in "Release key" bar.




- 2 Touch "Confirm."

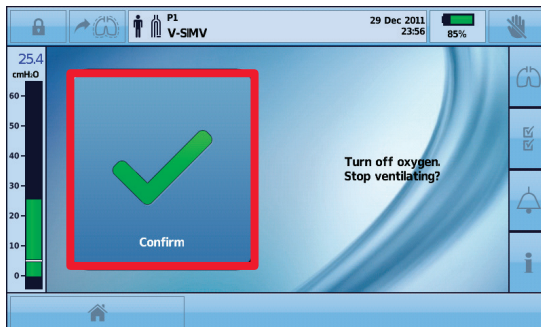

Getting to the home screen on Astral

- 1 Touch this tab to return to the home screen.

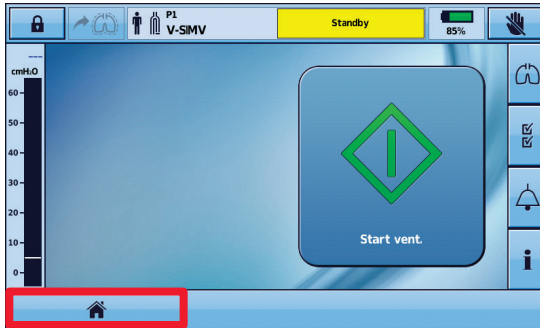



Distributed by ResMed Corp, 9001 Spectrum Center Boulevard, San Diego, CA 92123 USA +1 858 836 5000 or 1 800 424 0737 (toll free). See ResMed.com for other ResMed locations worldwide. Astral is a trademark and/or registered trademark of the ResMed family of companies. Specifications may change without notice. For patent and other intellectual property information, see ResMed.com/ip. © 2019 ResMed. 10111267/1 2019-04

[ResMed.com](https://www.resmed.com)